Antimicrobial Dermal Template

EN Product description

Endoform® Antimicrobial Dermal Template is an advanced wound care dressing primarily composed of natural, non-reconstituted collagen retaining the native extracellular matrix associated macromolecules including fibronectin, glycosaminoglycans, laminin, and elastin. The Dermal Template is supplied as sterile intact or fenestrated sheets.

Endoform® Antimicrobial Dermal Template contains approximately 12 µg/cm² (~0.3% w/w) ionic silver, intended to inhibit colonization of the dressing. The dressing is effective against a broad spectrum of microbes, including *Acinetobacter baumannii*, *Candida albicans*, *Candida parapsilosis*, *Candida glabrata*, *Escherichia coli*, Methicillin Resistant *Staphylococcus aureus* (MRSA), coagulase-negative *Staphylococci*, group A (beta-hemolytic) *Streptococci*, *Pseudomonas aeruginosa*, *Aspergillus niger*, and *Vancomycin Resistant Enterococci* (VRE). The dressing provides sustained antimicrobial effectiveness within the dressing for up to 7 days.

Endoform® Antimicrobial Dermal Template is derived from ovine (sheep) extracellular matrix and it retains the innate biological structure of the native extracellular matrix. When rehydrated with exudate or sterile saline, the Dermal Template transforms into a soft conforming sheet, which is naturally incorporated into the wound over time.

Not made with natural rubber latex.

Intended use

Endoform® Antimicrobial Dermal Template is a sterile, single use ovine forestomach-derived extracellular matrix intended to cover, protect, and provide a moist wound environment.

Indications for use

Endoform® Antimicrobial Dermal Template is indicated for the management of wounds including:

- Partial and full thickness wounds
- Pressure ulcers
- Venous ulcers
- Diabetic ulcers
- Chronic vascular ulcers
- Tunneled/ undermined wounds
- Surgical wounds (donor sites, grafts, post Moh's surgery, post laser surgery, podiatric, wound dehiscence)
- Traumatic wounds (abrasions, lacerations, first and second-degree burns, and skin tears)
- Draining wounds

Rx Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.

Contraindications

- **Endoform® Antimicrobial Dermal Template** is derived from an ovine (sheep) source and should not be used on patients with known sensitivity to ovine (sheep) derived material or ionic silver.
- **Endoform® Antimicrobial Dermal Template** is not indicated for use on third degree burns.

Precautions

- **Endoform® Antimicrobial Dermal Template** may be used when visible signs of infection are present in the wound area, only

when proper medical treatment addresses the underlying cause. Endoform Antimicrobial® Dermal Template is not intended to be a substitute for appropriate treatment of infection.

- **Endoform® Antimicrobial Dermal Template** is supplied as a sterile dressing. Do not use if the pouch seal is broken.
- Discard the device if mishandling has caused possible damage or contamination.
- Single use product. Do not attempt to re-sterilize. Discard all unused portions. Do not reuse.
- **Endoform® Antimicrobial Dermal Template**, like other silver-containing products, may cause transient discoloration of the wound bed and surrounding skin. Frequent or prolonged use of this product may result in permanent discoloration of the skin.
- Reduction in colonization or microbial growth on the device has not been shown to correlate with a reduction in infections in patients. Clinical studies to evaluate reduction in infection have not been performed.
- Healthcare Professionals should be aware that there is limited data on prolonged and repeated use of silver containing dressings, particularly in pediatrics and neonates.

Storage

Endoform® Antimicrobial Dermal Template should be stored between 15°C/59°F–40°C/104°F in a clean and dry area, away from direct light.

Appearance

Endoform® Antimicrobial Dermal Template, like other silver-containing products, may darken upon storage, after hydration in saline, when exposed to light, or when in contact with body fluids and tissues. This darkening does not affect product performance.

Directions for use

These recommendations are designed to serve only as a general guideline. They are not intended to supersede institutional protocols or professional clinical judgment concerning patient care.

1) Wound Preparation

- Prepare the wound bed by cleansing, irrigation and if necessary debridement to ensure the wound is free of debris, necrotic tissue or infected tissue.

2) Dressing Application

- Select a sheet of the dermal template which is slightly larger than the wound and apply aseptically.
- Endoform® Antimicrobial Dermal Template** can be applied as a whole sheet or trimmed so that it contacts the wound margins. Multiple sheets can be used to cover the entire wound bed.
- For ease of handling, apply the dermal template by placing the dry material in the wound and rehydrating with exudate or sterile saline. When hydrated **Endoform® Antimicrobial Dermal Template** transforms into a soft conforming sheet. Ensure that the dermal template conforms to the underlying wound bed.
- To protect **Endoform® Antimicrobial Dermal Template** from adhering to the secondary dressing, consider applying a non-adherent dressing over the dermal template to help protect the tissue while facilitating an optimal moist wound healing environment.
- Secure the dressing using an appropriate secondary dressing. The overlying dressing should be changed according to standard of care and taking into account the level of exudate.

- The biodegradable **Endoform® Antimicrobial Dermal Template** is naturally absorbed and incorporated into the wound over time. **Endoform® Antimicrobial Dermal Template** may last up to 7 days in the wound, depending on the wound environment.

3) Dressing Changes

- Duration of treatment and reapplication is determined by the physician and depends upon the wound type and conditions.
- Reapply when **Endoform® Antimicrobial Dermal Template** has been incorporated into the wound. It is recommended to assess the wound within 72 hours. **Endoform® Antimicrobial Dermal Template** may last up to 7 days in the wound, depending on the wound environment
- Carefully cleanse the wound surface in accordance with established procedures. Do not attempt to remove the off-white to golden gel that forms from residual **Endoform® Antimicrobial Dermal Template** in the wound as it contains extracellular matrix components.
- If areas of the dermal template are dry upon inspection, rehydrate with sterile saline and leave in place
- It is not necessary to remove any residual **Endoform® Antimicrobial Dermal Template** during dressing changes. However, if the product has been overlapped onto the peri-wound area, the remaining loose product that has not incorporated into the wound may be gently removed around the edges if desired.
- Change the secondary dressings as needed, and when **Endoform® Antimicrobial Dermal Template** is re-applied.

Symbols

ISO 15223-1: 2016 Medical devices – Symbols to be used with medical device labels, labeling and information to be supplied – Part 1: General requirements		
	5.4.2	Do not reuse
	5.2.6	Do not re-sterilize
	5.2.8	Do not use if package is damaged
	5.1.4	Use by expiration date YYYY/MM/DD
	5.3.4	Keep dry
	5.2.3	Sterilized with ethylene oxide
	5.3.7	Store at room temperature
	5.4.3	Consult instructions for use
	5.3.2	Keep away from heat and light
	-	Number of Units

Instructions for use

Antimicrobial Dermal Template

endoform

AROABIOSURGERY LTD.
2 Kingsford Smith Place,
Airport Oaks, Auckland 2022, New Zealand
1-860-337-7730

www.aroabio.com

Endoform® is a registered trademark of **Aroa Biosurgery Ltd.**

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©2020 Aroa Biosurgery Ltd